# CONFETTI/ CONFETTI TT

PROSTHETIC COVER





# CONFETTI/CONFETTI TT

PT	MANUAL DO USUÁRIO .....	5
EN	USER MANUAL.....	12
ES	MANUAL DEL USUARIO .....	20
FR	MANUAL DE L'UTILISATEUR.....	27
IT	MANUAL DELL'UTENTE.....	35
DE	BEDIENUNGSANLEITUNG .....	43
NL	GEBRUIKERSHANDLEIDING .....	51
UA	ПОСІБНИК КОРИСТУВАЧА .....	59


reddot award 2017  
best of the best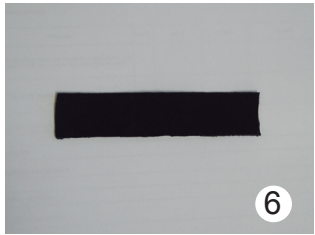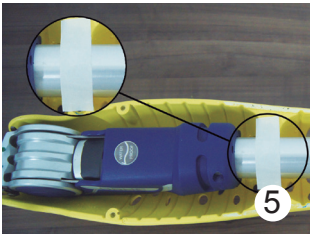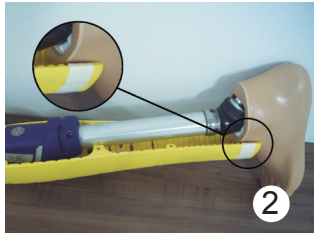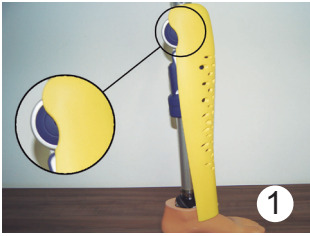

7

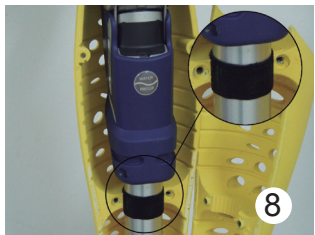**INFORMAÇÕES:**

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### **INFORMATION:**

LAST UPDATE: 10/04/2018.

- Read this manual carefully before using the product;
- Check the safety information to avoid injury and damage to the product;
- Instruct the user and the companion on the correct and safe use of the product;
- Save this document for future reference;
- If you have any questions, please do not hesitate to contact **ID ETHNOS** through the contacts informed at the end of this manual;
- Technical assistance and/or repairs may only be carried out through **ID ETHNOS** accredited representative or directly through **ID ETHNOS**.

## 1. PRODUCT DESCRIPTION

The prosthesis cover - **CONFETTI** is intended exclusively for patients who use a prosthesis for transfemoral amputation (above knee), disarticulated hip and

disarticulated knee, while the prosthesis cover - **CONFETTI TT** is intended for patients with prosthesis for transtibial amputation (below knee).

**ATTENTION:** the **CONFETTI** and the **CONFETTI TT** are intended for adults.

### 1.1 TECHNICAL SPECIFICATIONS

- Made in Polyurethane;
- Bipartite;
- Adaptable to patient's height and easily adjusted;
- Symmetrical format;
- Unisex;
- Customizable.

### 1.2 ACCESSORIES

- Instructions manual;
- Complete cover;
- 02 Adhesive tapes;
- 02 Ring clips;
- 02 Screws panela RM PN Phillips M4-0,70 X 30MA FE. BC;
- Cotton handbag.

Important: Only use original accessories with your **CONFETTI** and **CONFETTI TT**.

### 1.4 AVAILABLE COLOURS

#### **CONFETTI**

- **ID-001** CONFETTI BLACK;
- **ID-002** CONFETTI WHITE;
- **ID-003** CONFETTI RED;
- **ID-004** CONFETTI GREEN;
- **ID-005** CONFETTI YELLOW;
- **ID-006** CONFETTI BLUE;
- **ID-007** CONFETTI ORANGE;
- **ID-008** CONFETTI PINK;
- **ID-009** CONFETTI LIGHT BEIGE;
- **ID-010** CONFETTI BEIGE;
- **ID-011** CONFETTI BROWN.
- **ID-014** CONFETTI BLACK GOLDEN GLITTER;
- **ID-015** CONFETTI GRAY.

## **CONFETTI TT**

- **ID-001-TT** CONFETTI TT BLACK;
- **ID-002-TT** CONFETTI TT WHITE;
- **ID-003-TT** CONFETTI TT RED;
- **ID-004-TT** CONFETTI TT GREEN;
- **ID-005-TT** CONFETTI TT YELLOW;
- **ID-006-TT** CONFETTI TT BLUE;
- **ID-007-TT** CONFETTI TT ORANGE;
- **ID-008-TT** CONFETTI TT PINK;
- **ID-009-TT** CONFETTI TT LIGHT BEIGE;
- **ID-010-TT** CONFETTI TT BEIGE;
- **ID-011-TT** CONFETTI TT BROWN;
- **ID-014-TT** CONFETTI BLACK GOLDEN GLITTER;
- **ID-015-TT** CONFETTI GRAY.

## **2. GENERAL WARNING**

The cover can be adjusted according to the patient's height as shown in this manual's item 7.

When opening your **CONFETTI/CONFETTI TT**, make sure it is in perfect condition. If you have any issues, please contact us immediately via the available channels.

### **2.1 WEIGHT LIMIT**

There is no weight limit for the cover.

### **2.2 ENVIRONMENTAL CONDITIONS**

Contact with water or excess moisture can damage the cover, therefore:

- a. It is not recommended to use the cover in rain, swimming pool or places with water. The cover does not provide waterproof protection, and therefore, oxidation of the components may occur, shortening the life of the product;
- b. If the cover gets wet, dry it as soon as possible.

### **2.3 TERRAIN**

Avoid using the cover in sand, loose soil or on loose gravel terrain. This may damage the components.

### **2.4 INCORRECT HANDLING OF PACKAGE**

Make sure that the packaging materials are out of the reach of children as there may be choking hazard.

## **3. COMPATIBILITY**

The **CONFETTI/CONFETTI TT** is available for most prosthesis models today. Consult your prosthetist for more information regarding compatibility.

#### **4. SAFETY INSTRUCTIONS**

- a. Use a 4 mm Philips wrench for the screws;
- b. The cut for height adjustment should be performed according to the patient's measurements as instructed in this manual's item 7.

##### **Caution!**

Contact with cold, heat, embers or fire.

Injuries (e.g. burns) caused by melting of the material and damage to the product.

- Keep the product away from open flames, embers and other sources of heat;
- Do not expose the product to temperatures above +80°C or below -20°C.

#### **4.1 USAGE**

- Examine the product for damages before use;
- Do not use the product in case of operation limitations;
- This product has no side effects;
- In case of irreparable damage to the structure of the product, such as cracks due to falls or impacts, its use should be immediately interrupted due to risk of cuts during its handling.

#### **5. MAINTENANCE**

- a. Inspect and keep the cover in perfect condition;
- b. At least once a year, take the cover to an authorized dealer for a full inspection;
- c. Clean the cover regularly as instructed in this manual's item 5.1;
- d. Repair or replace worn or broken parts before use;
- e. If there is a need for more specific maintenance, contact your authorized dealer.

#### **5.1 CLEANING**

##### **5.1.1 COVER**

- a. Clean with a dry cloth, whenever necessary.

##### **5.1.2 SCREWS**

- a. Remove any fluff, dust or dirt on the screws.

#### **5.2 STORAGE**

- a. Store your cover in a clean, dry area, avoiding oxidation of some components;
- b. Before using it, make sure it is fully functional. Inspect it.

### **\*DISPOSAL INFORMATION\***

**CONFETTI/ CONFETTI TT** must be disposed in accordance with the environmental regulations of your region.

## **5.3 MAINTENANCE ITEMS**

### **5.3.1 SCREWS AND MAGNETS**


Always check the screws and magnets. If they are not in perfect condition, they will damage the structure and impair the cover's performance.

### **5.3.2 STRUCTURE**


Inspect the cover structure every 3 months and check for cracks or notches.

## **6. KNOW YOUR CONFETTI/CONFETTI TT**

### **6.1. CONFETTI**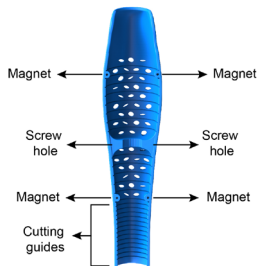


**INTERNAL FRONT VIEW**

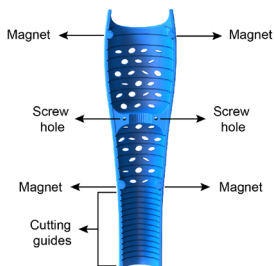
**INTERNAL BACK VIEW**

### **6.2. CONFETTI TT**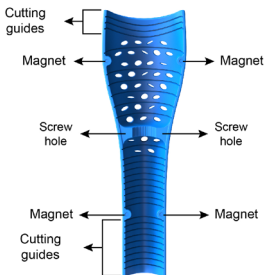

**INTERNAL FRONT VIEW**

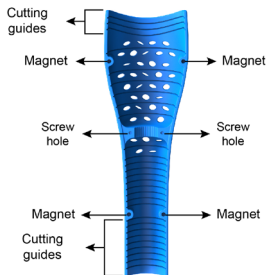**INTERNAL BACK VIEW**

## **7. HOW TO PERFORM THE ADJUSTMENT OF YOUR COVER?**

1. Make sure the cover is aligned close to the knee (**CONFETTI**) or prosthesis tube (**CONFETTI TT**) in a way that it does not interfere with the functioning of the components (Image 1);
2. Trace the place where it should be cut and make a straight cut with an object such as: knife, saw, stiletto, among others; (Images 2, 3 and 4). Do not cut the **CONFETTI** and leave it too close to the upper base of the foot, as it can disrupt the movement. We advise that it has a slack of 5mm;
3. Attach both parts of the cover to delimit the place where the back should be cut. Perform the cut in the same manner as in the previous step;
4. Fit the cover on the prosthesis to correctly mark the location where the tube will be located. (Image 5);

For the **CONFETTI**, use one of the accessories located in the kit that accompanies the package. If one of the accessories is placed improperly, the **CONFETTI'S** attachment may become unstable.

**4.1.** Adhesive tape – for 34 mm tubes. (Image 6);

**4.2.** Ring clip – for 30 mm tubes (thin tubes) (Image 7);

Check an example on how to use the accessories on image 8.

**5.** Attach the cover to the prosthesis using the magnets and screws.

### **Caution!**

- \* After the cover is cut, there is no possibility to glue or readjust it.
- \* We advise you to find a qualified professional to carry out the cutting adjustments.
- \* When placing the ring clip on your prosthesis' tube, be careful not to open it too much or it may break.

## 8. TECHNICAL SPECIFICATIONS

### 8.1 CONFETTI

<b>IMPORTANT:</b> * The measurement units are displayed in cm and g.	
<b>SPECIFICATIONS</b>	<b>MEASUREMENTS</b>
Frontal height (cm)	48
Overall frontal width (cm)	11
Frontal part depth (cm)	6
Rear part height (cm)	40
Rear part depth (cm)	5
Total weight (g)	approx. 585

### 8.2 CONFETTI TT

<b>IMPORTANT:</b> * The measurement units are displayed in cm and g.	
<b>SPECIFICATIONS</b>	<b>MEASUREMENTS</b>
Frontal height (cm)	40
Overall frontal width (cm)	12,6
Frontal part depth (cm)	6,5
Rear part height (cm)	12,6
Rear part depth (cm)	7
Total weight (g)	approx. 520

## 9. LEGAL

### 9.1 WARRANTY

1. The manufacturer warrants this device from the date of purchase. The warranty covers defects that can be proven to be a direct result of flaws in the material, production or construction and that are reported to the manufacturer within the warranty period. The decision on replacement or repair of defective parts will be made according to technical criteria. The warranty does not include normal equipment wear or improper cut. This warranty covers defects that occur under normal conditions. In regard to repair or replacement of parts that are rendered useless or damaged, expenses such as freight, stays or technicians are not included;

2. The buyer is aware of the provided manual and is obliged to comply with the items regarding the conservation and safety precautions prescribed, during the warranty period and after, even if the equipment is not in use. At request, the buyer will receive free copies of this manual;

3. The buyer is aware that the manufacturer's maintenance services are available;

4. When a problem is found, the customer should immediately contact the nearest service centre for warranty service

The guarantee shall cease when:

- There is result from negligence, acts of vandalism, inappropriate use or in disagreement with **ID ETHNOS** recommendations;
- There is absence of recommended maintenance, equipment or part modification or misuse;
- The equipment is damaged accidentally;
- The maintenance does not follow the orientations from this manual;
- There is use of non-original parts;
- There is damage due to abnormal environmental conditions such as: excessive temperature, excessive humidity, dust, gases, direct sun, rainfall and flood.
- The cover, or part of it, is stored in an inappropriate place exposed to weather conditions;
- **CONFETTI/CONFETTI TT** undergoes an improper adjustment process.

The warranty only applies when the product is used under the specified conditions and for the purposes for which it is intended

Do not use or install any third-party component on your cover that is not **ID ETHNOS** approved.

## 9.2 CE CONFORMITY

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outline in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

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Obelis S.A